



BOVINE POSTMORTEM – NEUROLOGIC DISEASE

Supplies:

- ◇ Screw top formalin jars (large)
- ◇ Whirlpak bags (one per tissue type)
- ◇ Sterile screw top tubes for fluids
- ◇ Sharpie marker to label bottles/bags

Important history to include: Age, commodity, treatments given, differentials, lesions seen on PM

Contact a pathologist:

519-824-4120 x 54530

Get post mortem supplies:

519-824-4120 x 54530

ahl.supplies@uoguelph.ca

Send in photos:

ahpath@uoguelph.ca

HISTOLOGY SAMPLES (FORMALIN)

Brain (one half side)	Liver
Pituitary	Kidney
Trigeminal ganglia	Spinal cord

If you suspect RABIES as a differential diagnosis, please consult OMAFRA's Rabies info page for step by step instructions including who to call/email, and when Public Health or OMAFRA should be involved. www.omafra.gov.on.ca/english/food/inspection/ahw/rabies.htm or google "OMAFRA Rabies")

MICROBIOLOGY (FRESH TISSUE)

Stomach contents	Brain (one half side) Virology	Liver Toxicology
Water Toxicology	Brainstem/meningeal swab Bacteriology	Kidney Toxicology
	Ocular fluid	Feed Toxicology