



BOVINE POSTMORTEM – DIARRHEA

Supplies:

- ◇ Screw top formalin jars (large)
- ◇ Whirlpak bags (one per tissue type)
- ◇ Sterile screw top tubes for fluids
- ◇ Sharpie marker to label

Important history to include:

Age, commodity, treatments given, differentials, lesions seen on PM

Contact a pathologist:

519-824-4120 x 54530

Get post mortem supplies:

519-824-4120 x 54530

ahl.supplies@uoguelph.ca

Send in photos:

ahlpath@uoguelph.ca

HISTOLOGY SAMPLES (FORMALIN)

Esophagus	Duodenum	Liver	Lung
Rumen	Jejunum	Spleen	Mesenteric Lymph Node
Reticulum	Ileum	Kidney	Cecum
Omasum	Spiral Colon	Abomasum	



BOVINE POSTMORTEM – DIARRHEA CONTINUED

Supplies:

- ◇ Screw top formalin jars (large)
- ◇ Whirlpak bags (one per tissue type)
- ◇ Sterile screw top tubes for fluids
- ◇ Sharpie marker to label bags

Important history to include:

Age, commodity, treatments given, differentials, lesions seen on PM

Contact a pathologist:

519-824-4120 x 54530

Get post mortem supplies:

519-824-4120 x 54530

ahl.supplies@uoguelph.ca

Send in photos:

ahpath@uoguelph.ca

MICROBIOLOGY (FRESH TISSUE)

Feces Bacteriology	Feces Virology (Rotavirus, coronavirus)	Feces Parasitology	Spleen Bacteriology
Ileum (Johne's suspect)	Spleen Virology (BVDV)	Mesenteric Lymph Node Virology (BVDV)	Kidney Bacteriology
Lung (Septicemia suspect)	Esophagus Virology (BVDV)	Liver Bacteriology	