

OAHN Swine Small Scale Herd Postmortem Project

Sudden Death in Sow / Gilt / Boar (1 animal per case)

TISSUE SAMPLING LIST

Fresh tissues: 1 sample per Whirl-Pak bag, per pig – label with stickers provided.

Formalin-fixed tissues: 1 sample jar per pig.

Tissue	Fresh (<i>include lesions</i>)	In Formalin (<i>include lesions</i>)
Tonsil	X	X
Thyroid gland		X
Trachea		X
Thymus		X
Lung	X 3	X
Heart		X
Liver	X	X
Spleen	X 2	X
Kidney	X	X
Adrenal gland		X
Pancreas		X
Mesenteric lymph node	X	X
Gastrohepatic lymph node	X	X
Stomach		X
Duodenum		X
Jejunum		X
Ileum		X
Spiral colon		X
Diaphragm		X
Skeletal muscle		X
Brain	X half, sagittally sectioned (OR entire head - fresh)	X half sagittally sectioned (OR entire head - fresh)



**ANIMAL HEALTH
LABORATORY**

Sudden Death in Sow / Gilt / Boar

Fresh tissues: Use a separate Whirl-Pak bag for each sample –label bags with the stickers provided (tissue name).

Formalin-fixed tissues: Use 1 sample jar per pig. For solid organs, samples should be approximately 1-2 cm². For intestine, take approximately 3 cm long segments and partially open longitudinally with scissors (to allow formalin to contact the mucosal surface).

Test list (for AHL use - included on submission form):

Virology: PRRSV PCR *prtqn*, influenza A virus PCR *inflpcr*: lung

Bacterial culture *bsetup*, *cults*: lung, spleen (tested separately)

Histopathology *hist*

Freeze and hold: tonsil / spleen, lung, kidney / liver, lymph nodes, brain



ANIMAL HEALTH
LABORATORY